

Viewing Reports in InteleBrowser

1. Log into InteleBrowser by going to <https://pacs.mdimaging.org>



- Once the radiologist has dictated a report, you can view this report in the browser. You can search for the patient by date, name, accession number, etc. Highlight the patient whose report you wish to view and click on the clipboard icon as shown below.

IntelPACS®

Patient Search

Patient ID: [Dropdown] Equals [Text Box]

Patient Name: [Dropdown] Begins [Text Box] Last Name, First Name

Accession Number: [Dropdown] Begins [Text Box] 04123142121390MDI

Search [Button] Reset [Button]

1 study, 1 selected

Patient Name	Patient ID	DOB	Sex	Acc. No.	Study Date	M. Spec. Study Description
TEST, JOHNNY==	2004123142121390MDI	1010 Jan 01	O	2004123142121390MDI	2004 Jan 23 14:21	...

1 series

Patient Name	Series Date	Mod.	Series No.	Images	Series Description	Source At
TEST, JOHNNY==	2004 Jan 23 14:21	CR	1	1	HAND PA	MEDPAC

- You can view the report on the screen or view it as a PDF and print if you wish by clicking on “View PDF” and printing.

IntelPACS®

View Report

You cannot make changes to this report because it was not transcribed through IntelPACS.

TEST, JOHNNY== (8 months other)

Patient ID: 2004123142121390MDI
 DOB: Mar 27, 2012
 Contact: [Text Box]

CR (1) - TESTING PS360 SIGNING

Referred by: ==, Study date: Jan 23, 2004 14:21
 Acc num: 2004123142121390MDI

Final Report

testing upper bunker.
 Electronically Signed by Amy Meyer on 3/27/2012 9:52 AM
 Dictated: Mar 27, 2012 09:52
 Transcribed: Amy Meyer (meyera)

Patient's Clinical History

View Images [Button] Add Addendum [Button] Revoke Signature [Button] Unlock Report [Button] Reload [Button] Delete [Button] **View PDF [Button]** Distribute [Button]